

3M Science.
Applied to Life.™



3M Casting and Splinting

When both sides are right, you can't go wrong.

When it comes to treating injuries, all splinting solutions are not created equal. Patients depend on you to provide the best possible care and support. That's why you should choose the splint that offers comfort and protection for patients and worry-free confidence for you.

Choose the new 3M™ Scotchcast™ Quick Step Double Sided Felt Roll Splint.

Breathable felt on both sides helps eliminate the risk of wrong-side application. No more guessing which side is right means no more mistakes. Give your patients the best, with help from 3M.

Introducing a two-sided solution from splinting experts.

FELT ON BOTH SIDES

Double-sided felt helps eliminate the risk of wrong-side application while providing padding and protection for the patient's injury.

WATER-REPELLANT TECHNOLOGY

Unlike splinting products that absorb water, the Scotchcast™ Quick Step Roll Splint sheds water, helping keep the patient's skin dry and reduce the risk of skin maceration.

CUSTOMIZABLE MOLD TIME

The resin inside the fiberglass responds based on the amount and temperature of the water. Using water that's 23°C/73°F provides 180 seconds (3 minutes) of molding time. To mold the splint faster, use warmer water.

CONFORMS EASILY TO BODY CONTOURS

The 7-layer fiberglass interior offers conformability around the bends and curves of the human body.

EASY-OPEN SEAMS ON BOTH SIDES

The felt can be easily opened on either side of the splint for quick access to the fiberglass layers and trouble-free trimming.

SETS IN 20 MINUTES

The Scotchcast™ Quick Step Roll Splint reaches full weight-bearing strength in just 20 minutes versus waiting a full day with a plaster product.

ROLL DELIVERY SYSTEM

Easy-to-use, easy-to-reseal packaging helps prevent exposure to moisture, which causes product hardening, and as a result, waste.

LIGHTWEIGHT, LOW-PROFILE CONSTRUCTION

Breathable and lightweight, yet strong and durable. Plus, the low profile means patients can easily fit their splint under a sleeve or jacket.



TRAINING AND EDUCATION

Our hands-on workshops and online video library help you learn proper splinting techniques so you can provide care with confidence.

Visit go.3m.com/castsplintvideos

Catalog Number	Description	Size	Packaging
74002Q	3M™ Scotchcast™ Quick Step Double Sided Felt Roll Splint	2 in x 15 ft.	1 roll/box 2 boxes/case
74003Q	3M™ Scotchcast™ Quick Step Double Sided Felt Roll Splint	3 in x 15 ft.	1 roll/box 2 boxes/case
74004Q	3M™ Scotchcast™ Quick Step Double Sided Felt Roll Splint	4 in x 15 ft.	1 roll/box 2 boxes/case
74005Q	3M™ Scotchcast™ Quick Step Double Sided Felt Roll Splint	5 in x 15 ft.	1 roll/box 2 boxes/case



3M Infection Prevention Division 3M Health Care

3M Center, 2510 Conway Ave
St. Paul, MN 55144-1000
U.S.A.

1 800 228 3957

www.3m.com/infectionprevention

3M and Scotchcast are trademarks of 3M.
Please recycle. Printed in the U.S.A.
© 3M 2015. All Rights Reserved.
70-2011-5671-1