



Read all instructions BEFORE
assembly and USE of product. KEEP
INSTRUCTIONS FOR FUTURE USE.

Lea todas las instrucciones ANTES
se armar y USAR este producto.
CONSERVE LAS INSTRUCCIONES
PARA USO FUTURO



Owner's Manual
Size 2, 3 & 4
Manual del Propietario

AXIOM ENDEAVOUR

INDOOR / OUTDOOR MOBILITY PUSH CHAIR

www.adaptivestar.com

1-800-278-9626



Failure to follow all warnings and instructions may result in serious injury or death.
For your passenger's safety, read the owner's manual before using the product.

- **PLEASE SAVE THE OWNER'S MANUAL FOR FUTURE USE.**
- **ADULT ASSEMBLY REQUIRED.**
- **NEVER LEAVE THE PASSENGER UNATTENDED.** Always keep passengers in view while in the push chair.
- **AVOID SERIOUS INJURY** from falling or sliding out. Always use 5-point harness. To get a snug fit, fasten the buckles and adjust the belts.
- **AVOID FINGER ENTRAPMENT:** Use care when folding and unfolding the push chair. Be certain that the push chair is fully open and latched before allowing your passenger near the push chair.
- **TO PREVENT A HAZARDOUS, UNSTABLE CONDITION,** never place purses, shopping bags, parcels or accessory items on the canopy or handle. Do not place more than 10 lbs. (4.5 kg) in the storage basket.
- **DO NOT** use storage basket as a child carrier.
- **DISCONTINUE USING YOUR PUSH CHAIR** should it become damaged or broken.
- **NEVER USE PUSH CHAIR ON STAIRS** or escalators. You may suddenly lose control of the product and your passenger may fall out. Take extra care when going up or down a step or curb.
- **BEFORE USE,** remove and dispose of all plastic bags and packaging materials and keep them out of reach of children.
- **DO NOT THROW AWAY BOX** chair came in, check that the product and all its components are all there and have not been damaged during transportation.
- **DO NOT** use accessories, spare parts, or any component not supplied or approved by Adaptive Star Mobility Inc. as they may make the push chair unsafe.
- **ALWAYS** use your safety wrist strap any time a passenger is in the push chair. Only attach wrist strap to the bottom of rear axle.
- **ALWAYS** engage rear foot brake when loading and unloading a passenger.
- **NEVER ALLOW PASSENGER TO STAND IN THE PUSH CHAIR.**
- **NEVER THROW AWAY THE SUPPORT BOARDS** that are in your seat or it could affect the balance. If they are poking out slightly it was solely for packaging purposes. Please tuck back into the seat and zip close.



El incumplimiento de todas las advertencias e instrucciones puede provocar lesiones graves o la muerte. Por la seguridad de su pasajero, lea las etiquetas y el manual del propietario antes de utilizar el producto.

- **CONSERVE EL MANUAL DE PROPIETARIO PARA USO FUTURO.**
- **SE REQUIERE EL ARMADO DE UN ADULTO.**
- **NUNCA DEJE AL PASAJERO DESANTENDIDO.** Cuando el pasajero esté en el cochecito, vigílelo en todo momento.
- **EVITE LAS LESIONES GRAVES** que podrían producirse a causa de la caída o deslizamiento del producto. Recuerde siempre utilizar el cinturón de seguridad. Después de sujetar las hebillas, ajuste bien los cinturones alrededor del pasajero.
- **EVITE LECCIONARSE LOS DEDOS:** Tenga cuidado al doblar y desdoblar la carriola. Asegúrese de que la silla de paseo esté completamente abierta y segura antes de permitir que su pasajero se acerque.
- **PARA EVITAR UNA INESTABILIDAD PELIGROSA,** nunca coloque carteras, bolsas de compras, paquetes o accesorios en la manija o en la capota. No cargue más de 10 libras (4.5 kg) en el cestillo portaobjeto.
- **NO** utilice el cestillo portaobjeto para transportar a un niño.
- **SUSPENDA EL USO DE LA CARRIOLA** si se daña o se rompe.
- **NUNCA UTILICE LA SILLA DE EMPUJE EN ESCALERAS,** ya sean comunes o mecánicas. Puede perder repentinamente el control de la silla de paseo su pasajero puede caerse. Asimismo, tenga extremo cuidado cuando suba o baje un escalón de vereda.
- **ANTES DEL USO,** retire y deseche todas las bolsa de plástico y los materiales de embalaje y manténgalos fuera del alcance de los niño.
- **NO DESECHAR LA CAJA** en la que vino la silla, verifique que el producto y todos sus componentes estén todos allí y no hayan sufrido daños durante el transporte.
- **NO** utilice accesorios, piezas de repuesto ni componentes que no hayan sido suministrados o aprobados por Adaptive Star Mobility Inc. Ya que podrían comprometer la estabilidad del cochecito.
- Utilice **SIEMPRE** su muñequera de seguridad cada vez que un pasajero esté en el cochecito. Coloque la muñequera únicamente en la parte inferior del eje trasero.
- **SIEMPRE** ponga el freno de pie trasero al cargar y descargar a un pasajero.
- **NUNCA PERMITA QUE EL PASAJERO SE PARE EN LA SILLA DE EMPUJE.**
- **NUNCA** deseche las tablas de apoyo que se encuentran en tu asiento o podría afectar el equilibrio. Si sobresalen ligeramente, es únicamente por motivos de embalaje. Por favor, recuéstese en el asiento y cierre la cremallera.

FREQUENTLY ASKED QUESTIONS

What if I break my Axiom Push Chair?

No problem! Just call us for help at 1-800-278-9626. We'll get you back up and on the road in no time.

Is the fabric on my Axiom Push Chair washable?

Yes! The seat, canopy, and basket are removable and machine washable in cold water with mild detergent. After washing, place the seat back onto the pushchair for drying. Machine drying is not recommended. Please remember to remove the support boards before washing and place them back on when finished drying.

Do I need to keep the support boards that come in my seat?

Yes! The support boards are extremely important for the balance of your pushchair. If they are sticking out, it is solely for packaging purposes. Push them back in and zip up your seatback.

Do I need to use the safety wrist strap?

Yes! It is extremely important that the safety wrist strap is used every time you use your push chair. This will ensure that you stay in control of your chair and will keep your passenger safe at all times.

How far does my Axiom Push Chair recline?

30 degrees, unless customized.

My Axiom Push Chair isn't tracking straight

Check the tire pressure on all three tires. Ensure that they are not too full or too flat. Be sure that the front wheel is not too tight on the forks. The wheel being too tight can cause the push chair to not track straight.

Can I roller blade with my Axiom Push Chair?

We do not recommend this: it is just too easy to lose control and cause possible harm to your passengers or yourself.

COMPONENTS

COMPONENTES



Folded Chair
Silla plegable



Support Boards
Tableros de soporte
****installed****



Rear Wheels
Ruedas traseras



Quick Release Axles
Ruedas traseras
****installed****




Front Wheel
Rueda delantera



Quick Release Axle
Ruedas traseras
****installed****



Safety Wrist Strap
Correa de seguridad
para la muñeca
****installed****

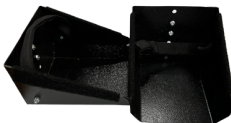

Foot Buckets
Espacios para los pies
****packed in small box in***
chair box with fender*



Fender
Defensa

LET'S GET STARTED!



Your Axiom Endeavour Pushchair is almost completely assembled when you receive it. The following short series of instructions will help you through the final steps so you can get out and start having fun!

Remove the Endeavour from the box, cut all white zip ties and unwrap all the component pieces. Please keep your box until you are sure you are keeping your chair and have confirmed that the content in the box matches the photo above. If support boards are poking out of seatback, please do not throw them away, they were pulled out only for packaging purposes. All you need to do is insert them back in the seat and zip it closed. Support boards must be left in your seat, as they affect the balance of your chair. If you are missing something, please call 1-800-278-9626 and talk with one of our helpful Technicians.

To unfold the Endeavour into its upright use position, you will just need to hold onto the adjustable handle while placing your foot on the rear brake, gently pulling up on the handle until you hear a click which lets you know the chair is secure.

Your Axiom Endeavour folds to a convenient size that will allow you to transport it in many automobile trunks. However, if you require a more compact folded size, simply remove the wheels.

The Endeavour push chair comes with a secondary safety measure. Please ensure that the safety pin is installed before loading the passenger.

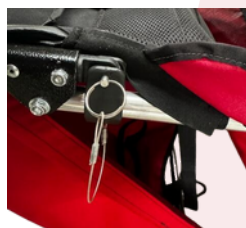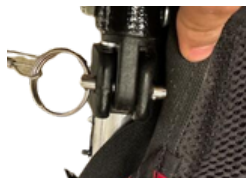

Safety Pin Installed



WARNING!!!

MAKE SURE THE UNIT IS UNFOLDED AND LOCKED INTO PLACE BEFORE LOADING THE PASSENGER

To install or remove the rear wheels simply push the button on the outside of the wheel and hold it in while either installing it into the Endeavour axle or removing it. Be sure the rear foot brake is in the upright position when installing or removing the wheels.



Inside of
Rear Wheel



Outside of
Rear Wheel



Big Button on
Quick Release Axle

REAR WHEEL INSTALLATION

- 1** First unlock the rear brake by Pulling the black loop upwards



- 2** Next, push the black button and while holding it in...



- 3** Insert the wheel axle all the way into the pushchair frame and release the button. Now repeat to install the other wheel.




**TO ENSURE WHEELS ARE LOCKED IN,
GIVE THEM A NICE TUG BEFORE USE**

Foot Bucket and Fender are in their own box packed inside the box with the chair.

FENDER INSTALLATION

Find the front wheel fender and install it onto the four bolt studs as shown in the photo. Note: the retaining nuts for these bolts have been installed for shipping. Please remove them for installing the fender. Once the fender is installed, tighten the nuts onto the bolts with a wrench. Tighten "Snuggly" being cautious to not over tighten them.

Mounting
Bolts are
indicated by
white arrows

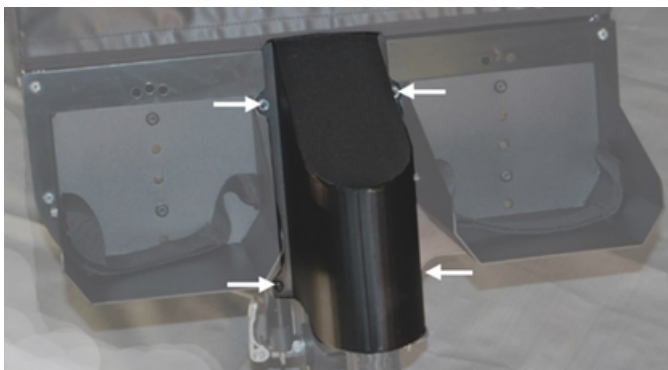


Use 10mm wrench
for tightening nuts

FOOT BUCKET INSTALLATION

Each foot bucket has the mounting nuts and bolts installed for shipping. Please remove them now and set them aside to be used when the foot bucket is installed. Each foot bucket is universal, meaning there is no right side / left side designation. Mount each foot bucket by bolting it onto the Endeavour as shown in the photo. On the Endeavour is a set of holes that will allow you to adjust the height and side to side angle of each footwell. You will need to determine the correct height and angle of you application. Please allow at least 1 empty hole between mounting bolts for strength and security.

Mounting
Bolts are
indicated by
white arrows

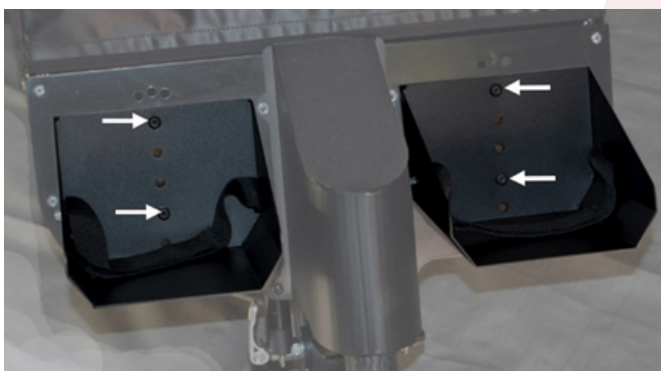

Use 7/32 Allen Wrench
to fasten bolt.

Use 7/16 Wrench to
hold nylon nuts in place
behind foot buckets

FRONT WHEEL INSTALLATION

- 1** Start by tipping the push chair backwards so the fork assembly is pointing upward. This will make it easier to install the wheel.



- 2** The front wheel is the same as a bicycle. Place the tire between the brake pads and orient your wheel so that the flat sections of the wheel axle align with the slotted metal fork tips and push the wheel so that it sits all the way in the back of the fork tip



- 3** Position the silver lever straight out from the frame as shown. Now tighten the silver knob on the opposite side of the lever until it just starts to become snug. Overtightening can cause tracking issues



- 4** Now swing the lever closed as shown here. If you are unable to swing the lever completely into the locked position, open it back straight out, loosen the silver knob slightly, and close the lever again.



ADJUSTABLE HEIGHT HANDLE

Your pushchair is equipped with a height adjustable handle. To adjust the handle to best suit your comfort, simply press the buttons located on the inside of the handle near the end of the foam handle grip. While holding the buttons in, adjust the handle up or down to the height that is correct for you. Once there, release the buttons and the handle will lock into place.



← Highest Position

← Middle Position

← Lowest Position

Pushbar height options

FOLDING

To fold your Endeavour, remove the safety pin from the secondary folding safety feature.



Safety Pin Removed
Must remove to fold

Unbutton the seat anchor strap located under the release lever, and locate the primary folding release lever located on the left side upright of the Endeavour.



If you are having trouble folding your Endeavour Pushchair, ensure that the folding brackets are not getting caught on the fabric of the seat.



SAFETY WRIST STRAP

Your Axiom pushchair is equipped with a safety wrist strap which you should always use when there is a passenger loaded. The wrist strap is the link between you and the pushchair and keeps the pushchair from rolling away out of your control by accident. Never leave a passenger unattended in the pushchair. The wrist strap is constructed with loops on each end. The wrist strap should be attached to the pushchair at the frame member that runs between the rear wheels, by looping around it, and inserting the one loop end through the other loop, pulling it tight. The loop may now be wrapped around your wrist attaching you to the push chair.



Safety Wrist strap needs to be on top of the Rear Brake



Safety Wrist strap with Rear Brake down



**NEVER ATTACH THE WRIST STRAP TO THE
ADJUSTABLE HANDLE
NEVER ATTACH THE WRIST STRAP TO AN ANIMAL**

ADJUSTMENTS

SEAT RECLINE

To recline the seat, simply slide the two straps behind the seat. To put the seat back to the upright position, pull the webbing taut until in the upright position.



FRONT BRAKE ADJUSTMENT

The front brake on your Endeavour push chair may require periodic adjustment. By turning the adjusting barrel, indicated by the red arrow, outward it will cause the brake pads to move closer to the front wheel. There is a similar adjuster located underneath the pushchair, near the front wheel, on the brake caliper itself. This caliper style brake system is the same as on a bicycle, and if you require more detailed assistance, a local bicycle shop will be able to provide it for you.



TRACKING

If your pushchair is pulling to the left or right while using the fix front wheel, check to see if it is one of the following:

Tire pressure, check to see if all 3 tires are fully inflated.

Be sure that the front wheel hasn't been installed too tight. Release the QR lever, push the front wheel all the way back into the fork tips and when closing the lever make sure that it is only snug, not HE-MAN tight. If it is, loosen the thumb nut on the other side and close the lever. You may have to make several small adjustments until the lever closes snugly.




BRAKING SYSTEM

Your push chair is equipped with a brake for the front wheel and a brake for the rear wheels. The front wheel brake ONLY works with the standard front wheel, It WILL NOT work with the accessory swivel wheel axis kit.

The rear brake works by pushing down on the back bar with the lock that runs between the rear wheels. It will lock into the black toothed hubs on the rear wheels.



Hand Brake

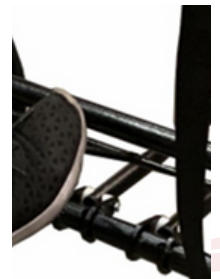

Foot Brake

TIRE PRESSURE

If your tire pressure is low the PSI is located on the side of the tire. A large difference can cause pulling to one side.



FOOT BUCKETS

Your Endeavour push chair features footwells that are independently adjustable to passenger comfort. Each footwell can adjust up and down in 1" increments.



HARNESS ADJUSTMENTS

You will want to do a onetime general positioning of the harness before placing the passenger in the seat. Once the passenger is in the seat then do the final harness adjustments as follows:

Undo both the buckles from the padded center pelvic strap



Pull up the webbing through the seat and up on the shoulder harness area.



Push up the black cam buckle located between the passenger's knees and pull the webbing out the end. Once it is at the desired length flip the cam buckle down to secure it into place.



Cam
Buckle
Up



Cam
Buckle
Down

Place the passenger in the seat.

Adjust the padded center strap forward or backward to fit your passenger.

Adjust the padded shoulder harness up or down on the Velcro strips to fit your passenger.

Buckle the two buckles to the padded center strap.

Pull the adjustment strap until the center strap and the shoulder harness are both snug.

When adjusting the harness, think of a baby car seat, the harness is very similar.

If the harness is too small for the passenger, harness extensions are available for purchase

ALWAYS CHECK THAT THE SEAT ANCHOR STRAPS ARE ATTACHED PROPERLY BEFORE EACH USE



**AVOID SERIOUS INJURY FROM FALLING
OR SLIDING OUT BY ALWAYS USING THE
5-POINT HARNESS SYSTEM**

MAINTENANCE

We recommend that you have an experienced dealer perform these maintenance check at least once each year. Lubrication of these following items is necessary:

- Pivot point and cable of hand brake
- Pivot point and cable of brake caliper
- Front wheel axle
- Front wheel quick release lever
- Insert in frame for rear wheels
- Wheel hub and bearings of rear wheels
- Locking bars of folding mechanism
- All folding joints of pushchair

Check to make sure all the bolts are tight and that springs are secure on the rear foot brake.

Check fabric seams and straps for fraying and tearing.

Keep pushchair out of the sun and in a dry place when not in use, to prevent mold and mildew.

Check the frame for loose rivets, cracks, and wear.

Check to make sure all nuts and bolts are tight on the frame.

We recommend that you put a rust preventative type coating on your wheels, nuts, bolts, brake springs and brake caliper on your pushchair. These parts can also be maintained by cleaning with a damp cloth and keeping these parts dry and free from debris.

The tires on your pushchair are a standard bicycle size and have an inner tube. If one of your tires will not hold air, the inner tube can be replaced and both tires and inner tubes can be found at a professional bicycle store and most major retailers.

The front brake uses a standard bicycle style caliper and brake pad. If replacement brake pads are needed, they can be found at professional bicycle stores and most major retailers.



**FAILING TO PERFORM MAINTENANCE CHECKS CAN
RESULT IN YOUR PUSHCHAIR NOT OPERATING
CORRECTLY AND MAY RESULT IN A VOIDED WARRANTY**



WARRANTY CARD

Thank you for purchasing an Adaptive Star Indoor/Outdoor Mobility Push Chair. We have created a warranty that will keep your unit in top condition and headache free. The following warranty is on workmanship and materials from the date of original consumer purchase:

- A 5-year warranty on the frame.
- A 1-year warranty on parts and soft goods. Exclusions include
- Parts that require replacement under normal use.
- Damage or malfunctions caused by negligence or abuse
- Foam grips, inner tubes, tires, and brake pads are specifically excluded.

Adaptive Star will, at its option, repair or replace defective parts upon delivery to our service center accompanied by proof of date of first consumer purchase, such as a copy of a sales receipt. Please refer to the store where you made your purchase for returns and refunds as Adaptive Star does not give direct refunds. Purchaser is responsible for initial shipping charges required to ship the product for warranty service. Purchaser is also responsible for all applicable duties and/or custom charges. The return charges will be at Adaptive Stars expense. Please include your name, address, telephone number, email address and a brief description of the problem. Adaptive Star does not accept delivery of any product without a Return Authorization# clearly marked on the return shipment box and shipping label.

All implied warranties, including warranties of merchantability and fitness for a particular purpose are limited in duration to the length of this warranty. Adaptive Star will not be liable for any incidental, consequential, or other damages, including, without limitation, damages resulting from loss of use. Always keep the box the chair came in until you know the chair is complete in working order. In the event of UPS damage call them so that a driver can come out and inspect the package. A claim cannot be file without the box the chair came in. Some states do not allow limitation on ow long a implied warranty last and/or do not allow the exclusions or limitations of incidental or consequential damages, so the about limitations my not apply to you. This limited warranty gives you specific legal rights, and you may also have other rights, which vary from state to state.

Please complete product warranty on our website. For any helpful hits or video instructions, please visit our YouTube channel.



ADAPTIVE STAR MOBILITY