Shower Bench

Banc De Douche

Banco De Ducha

Instructions For Use
Instructions D'utilisation
Instrucciones De Uso

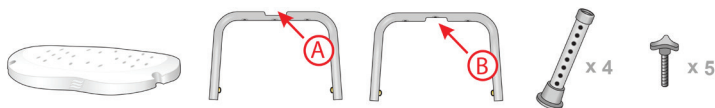
Item # / Article n° / Artículo n.º: MD770535WH

VER.A.10.25


Assembly Instructions

- 1** Remove contents from carton.


- 2** Place seat upside down on flat surface.

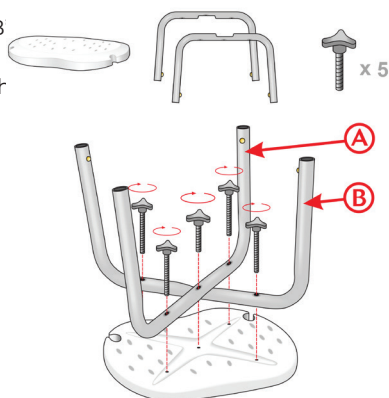- 3** Place the leg section marked with a "B" into the groove of the bottom of the seat. Place the leg section marked with an "A" on top of the "B" tube and into the groove. Insert the thumb screw through the middle hole. Insert the remaining 4 thumb screws through the remaining 4 holes.

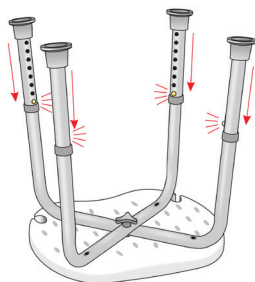- 4** Insert legs into leg frame by depressing push pins, sliding legs over the receptacle and setting to desired height. Make sure the push button protrudes fully through the receptacle hole. Repeat for remaining legs.


Assembly Instructions

- 5** Turn bath bench over on it's legs.

Adjusting Leg Height

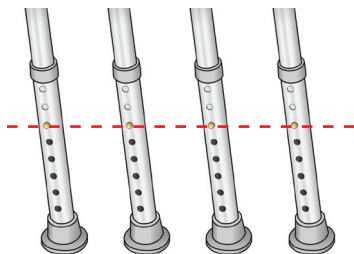- 1** Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes.
- 2** To be sure legs are locked into place by pushing down on or by pulling legs.

Precautions

WEIGHT CAPACITY: 300 LB / 136 KG

- 1** Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
- 2** Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
- 3** Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
- 4** All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
- 5** DO NOT stand on chair.

NOTE: To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

Maintenance and Cleaning

Brush off dust and dirt or wipe with a damp cloth. Clean regularly by hand with a damp cloth and mild detergent only. Do not use aggressive detergents such as chlorine, bleach or abrasive cleaners. M+D by Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, loose, or rusted, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit www.drivemedical.com/help for more information.

Lifetime Limited Warranty

Your M+D by Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

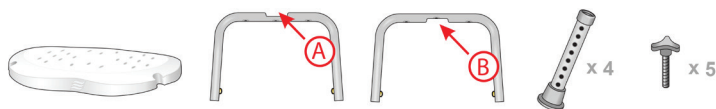
This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your M+D by Drive device or this warranty, please contact an authorized Drive dealer.

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
This image shows a single sheet of white paper with horizontal blue or grey ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

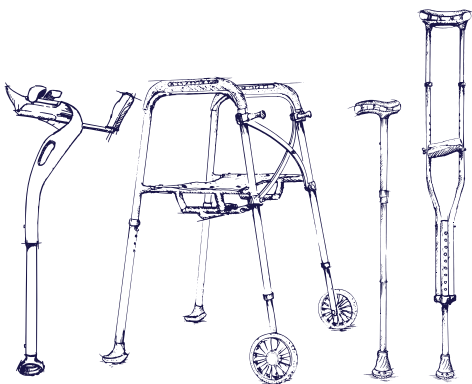[illegible]

© 2025 Medical Depot, Inc. dba Drive DeVilbiss Healthcare.
All rights reserved.

Mobility Designed® is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA
Made in China

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© 2025 Medical Depot, Inc. exerçant son activité sous le nom
Mobility Designed® est une marque déposée de Medical Depot, Inc.
Mobility Designed® es una marca registrada de Medical Depot, Inc.