MOBILITY DESIGNED

Massage Bath Mat

Tapis De Bain Massant

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Instructions For Use
Instructions D'utilisation
Instrucciones De Uso

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



VER.B.12.25

Instructions

- 1** Rinse the mat thoroughly with warm water to remove any dust or packaging residue.
- 2** Clean the bath or shower surface where the mat will be placed. Lightly spray the area with water to help the suction cups adhere better.
- 3** Place the mat in your desired location with the suction cups facing down.
- 4** Press down firmly on the entire surface of the mat, focusing on the corners and edges, to activate the suction cups and ensure a secure grip.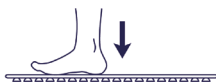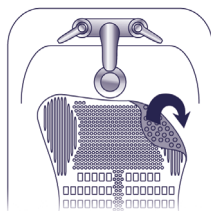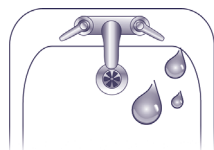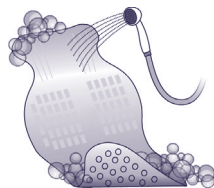

Additional Precautions

- 1** Always ensure bath mat is properly secure to bathtub before use.
- 2** Regularly clean bathtub to avoid buildup of slippery residues.
- 3** Do not use bleach or strong abrasives when cleaning your bath mat.
- 4** Ensure all cleaning products are thoroughly rinsed off bathtub before using bathmat.

Maintenance and Cleaning

- 1** Rinse both sides of bath mat after each use.
- 2** Remove and hang dry after each use to maximize product lifespan.
- 3** Regularly machine wash bath mat in delicate cycle in warm/cold water.
- 4** Hair dyes and colorings may permanently stain or discolor the rubber.

Lifetime Limited Warranty

Your M+D by Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your M+D by Drive device or this warranty, please contact an authorized Drive dealer.

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All rights reserved.
Mobility Designed® is a trademark of Medical Depot, Inc.
© 2025 Medical Depot, Inc. dba Drive DeVilbiss Healthcare.
© 2025 Medical Depot, Inc. exerçant son activité sous le nom
Mobility Designed® est une marque déposée de Medical Depot, Inc.
Mobility Designed® es una marca registrada de Medical Depot, Inc.