# **Nitro Cruise Wheelchair**

## **Fauteuil roulant Nitro Cruise**

### **Silla de ruedas Nitro Cruise**

item #: / n° d'article : / N.° de artículo:

NW16FBADARD-SF  
NW18FBADARD-SF  
NW20FBADARD-SF  
NW16FBADABK-SF  
NW18FBADABK-SF  
NW20FBADABK-SF  
NW16FBADARD-ELR  
NW18FBADARD-ELR  
NW20FBADARD-ELR  
NW16FBADABK-ELR  
NW18FBADABK-ELR  
NW20FBADABK-ELR

**WEIGHT CAPACITY**  
**16", 18" AND 20" CHAIRS: 300 LB (136 KG)**

## **Intended Use**

The wheelchair is a manually propelled mobility device intended to provide safe, supportive seating and mobility assistance for individuals with temporary or permanent mobility limitations. It is designed to enable users to move within their home or community environments, with support from a caregiver or through self-propulsion, depending on the model.

The device is intended to assist individuals who require postural support, mobility assistance, or reduced walking effort due to injury, illness, disability, or age-related conditions. It is suitable for use in indoor and outdoor settings on stable, level surfaces.

The wheelchair is not intended for use as a seating system in motor vehicles.

## CAUTION:

**Stability** - Wheelchair is designed to be safe and stable as long as center of gravity is maintained while going about normal activities. Care must be taken while performing activities that shift your center of gravity in the chair. Never reach so far that it requires you to shift in the seat or bend over out of the chair. When you have to reach for objects that require you to lean, be certain front casters are pointing in a forward position that extends the wheel base and will stabilize the chair.

**Ramps** - Be sure of your own capabilities, limitations and strength before attempting to go up a ramp. Correct techniques on ramp negotiation should be shown to you by a qualified professional. Assistance is recommended when going up or down inclines greater than 10°. If you must stop on an incline, avoid sudden, abrupt movement when you resume movement. Because of backward angle of the chair, sudden abrupt movement could cause instability. Attempting to use the wheel lock brakes could result in accidental locking, tipping to one side, or cause a sudden stop. Avoid changing direction when going down an incline, as this could result in instability of tipping over.

Please read this manual carefully to ensure safe usage and understanding of maintenance instructions.

*Congratulations on your purchase of a Drive Medical wheelchair. Drive Medical wheelchairs are the result of quality engineering and are manufactured to meet our highly rigorous specifications. We hope it will surpass all your expectations.*

## Use and Safety

**WHEEL LOCKS** - Engage both wheel locks before getting in or out of chair, leaning forward in chair, or while in an elevator or wheelchair lift. Proper adjustment of wheel lock is at least 1/8" when locked. Locks are engaged by pushing handle completely forward. Adjustments to locks are made by loosening or tightening nut on the carriage bolt. Embed lock shoe at least 1/8" by sliding clamp toward rear wheel while handle is engaged in locked position. Tighten nut and bolt to secure in position. Test for correct locking action before actual use.

**TIRES** - Solid rubber tires need minimal maintenance, but should be cleaned occasionally with a damp cloth. Replace if tires become severely worn or cracking appears.

**FRONT CASTERS** - Front casters and associated mounting hardware should be inspected at least once a month to ensure proper tightness. Lubricate the casters by removing fork and greasing caster stem bearings. Lubricate wheel bearings by removing wheel from fork. Reassemble fork and wheel by tightening nuts until no play remains, but wheels roll freely. If nut is too loose, caster will shimmy, if too tight, chair will be hard to steer.

**REMOVAL OF SWING-AWAY FOOT RESTS/LEG RESTS** - Release lock by pulling up on release lever. Swing-away foot rest /leg rest out and sideways. Lift to remove.

**WHEELS** - Large rear wheels should be inspected at least once a month to ensure proper tightness. If wheel wobbles or is loose, release lock nut and tighten axle bolt slightly. Test for proper tightness by leaning chair to one side and spin adjusted wheel.

**GENERAL CARE** - Wipe parts with a clean, soft cloth at least once a week. All metal parts can

be cleaned with auto wax or similar products. If exposed to moisture, dry immediately. Do not use an abrasive cleanser that will scratch the finish.

**TO OPEN CHAIR** - Face the front of the chair, place hands on seat rails and push down and out, as the seat opens until level.

**TO CLOSE/FOLD CHAIR** - First fold foot plates to vertical position. Grab seat with both hands at front and back edge and lift. Or, tilt chair to one side and lift up on seat rail.

**WARNING**

Do not attempt to adjust or operate wheelchair without reading carefully all instructions in this manual.

This wheelchair has not been approved as a seat that can be used within vehicles of any kind. Always transfer wheelchair user to a vehicle manufacturers approved seat and use safety restraints available.

Do not operate this wheelchair on roadways, streets or any surfaces with vehicular traffic.

Do not attempt to operate on inclines 10% or greater without proper assistance.

Be sure to completely engage wheel locks when getting in and out of wheelchair or when in elevator or wheelchair lift.

Wheelchair should be on a smooth, stable and level area with wheel locks engaged before transferring user in or out of wheelchair.

Do not adjust foot rests any lower than 2-1/2" from floor.

When getting in or out of wheelchair, do not stand on or apply weight to foot rests.

Before reaching or leaning forward, rotate casters to front of the wheelchair engage wheel locks to help prevent wheelchair from tipping over.

Do not lean over or apply weight to the top of wheelchair back. This can cause wheelchair to tip over. When trying to extend arms over back, only reach as far as your arm can extend without having to alter your seating position.

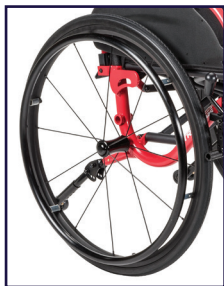
DO NOT stand on wheelchair.

DO NOT use wheel locks to slow your descent down an incline.

Abruptly changing directions while going down an incline could cause instability or possible tipping over.

## Rear Wheel Positioning / Seat Height Adjustment

The Nitro Cruise comes with a quick release rear axle. To remove the rear wheel, depress the center hub and the lock will disengage, allowing the wheel to be removed.

To adjust the axle position and lower the wheelchair seat to floor height, follow the below steps:

### **1. Remove the Wheel**

Use the quick-release axle to detach the wheel from the frame.

### **2. Locate the mounting insert**

The black mounting insert is positioned inside the lower mounting hole on the wheelchair frame (as shown in the photo).

### **3. Remove the Black Insert**

Use one wrench to loosen the insert. Use the second wrench to hold the opposite side in place during removal to prevent it from spinning.

### **4. Reinstall the Insert in the Top Hole**

Move the black insert to the top hole on the frame to raise the seat height. Secure it tightly using the same two-wrench method.

### **5. Reattach the Wheel**

Insert the quick-release axle back through the wheel hub and into the relocated mounting insert. Ensure the wheel is locked securely.

## **IMPORTANT: ANTI-TIPPER AND FRONT CASTER ADJUSTMENT REQUIRED**

Whenever the seat height is adjusted, the anti-tippers and front caster position must also be checked and adjusted as needed to maintain proper wheelchair stability and handling.

Changing the seat height alters the wheelchair's center of gravity and wheel geometry. Proper anti-tipper and front caster positioning is required to help reduce the risk of tipping and ensure smooth operation.

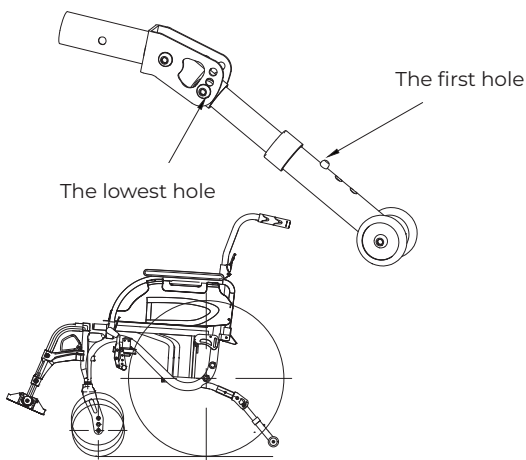
### **Anti-Tipper Positioning After Seat Height Adjustment**

The anti-tippers must always be installed at the same relative height as the seat.

Refer to the accompanying instructions and illustrations showing the lowest hole and upper hole positions.


After completing the rear wheel and seat height adjustment:

- If the seat is installed using the lowest frame mounting hole, install the anti-tippers in the lowest mounting position.

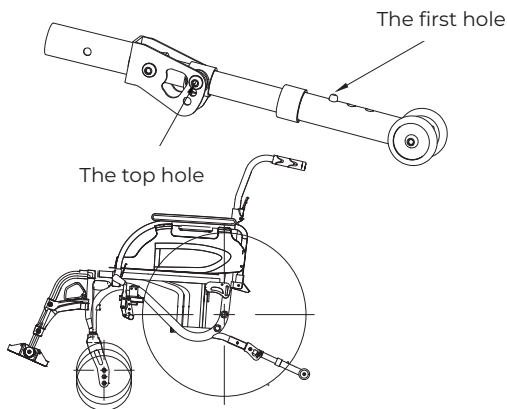- If the seat is installed using the upper frame mounting hole, install the anti-tippers in the upper mounting position.

## Front Caster Positioning After Seat Height Adjustment

After changing the seat height, verify that the front casters are set in the appropriate position to align with the rear wheel height.

Adjust the front caster mounting position as needed to maintain proper frame level and steering performance.

Front casters should be set so the wheelchair sits level and rolls smoothly.

Refer to the Front Caster Positioning section of this manual for detailed adjustment instructions.

## Front Caster Positioning

The 3 position front caster can be adjusted for positioning with the rear wheel. To adjust the front caster, loosen the bolt (outside of the caster) with an allen wrench (included). Remove the bolt and axle and place into the desired position. After the axle is placed in the new position, turn the allen screw clockwise to tighten.

## Adjusting Front Riggings

To extend the length of the front riggings, loosen the bolt located above the heel loops. Slide the tubing to the desired height and tighten bolt.

## Adjusting Footplate Angle

To adjust the angle of the footplate, loosen the bolt located at the pivot point where the footplate connects to the footrest.

Once the bolt is loosened, rotate the footplate to the desired angle and then securely re-tighten the bolt before use.

The footplate angle adjustment range is 9° to 37°.

## Adjusting Arm Rest

**To adjust the height of the arm rest** - Swing back armrest and remove the two bolts beneath the armrest assembly. There are two height adjustment blocks located beneath the armrest frame. To raise the armrest height, reposition either one or both blocks to the top of the armrest frame directly beneath the armrest pad. Replace the bolts beneath the armrest frame to secure into place.

Height Adjustment  
Blocks

Bolts

Armrest  
Pad

## Back Cushion

Ensure that removable back cushion is securely attached to the back upholstery by aligning the hook-and-loop fasteners.

The removable back cushion is machine washable. To clean the cushion, wash on a gentle cycle with cold water and a mild detergent. Do not use bleach or fabric softener. Air dry completely before reattaching to the wheelchair. Do not machine dry.

Foam positioning blocks may be attached to the inside surface of the back cushion to provide additional lateral or posterior support as needed. To install, position the blocks in the desired location and attach using the integrated hook-and-loop fasteners. Ensure each block is firmly secured before use.

Positioning blocks are not machine washable. To clean, wipe the foam surface with a mild soap solution and a damp cloth. Allow the blocks to air dry completely before reattaching. Do not soak, machine wash, or machine dry the positioning blocks.

## Anti-tippers

To install the anti-tippers, locate the rubber caps on the rear lower frame of the wheelchair and remove them.

Position each anti-tipper over the rear lower frame tube so that the mounting bracket encloses the frame. Slide the anti-tipper assembly into place until the detent button engages and locks securely around the frame.

Ensure both anti-tippers are firmly attached and oriented downward before using the wheelchair.

## Seat Extension Instructions

Your Nitro Cruise Wheelchair comes with an adjustable depth seat that has been factory set at the shortest position.

Follow the instructions below to extend the seat depth of your chair.

1. Lay wheelchair on its back. Open seat two-thirds of the way so seat posts do NOT rest on chair frame. Unfold seat extension by pulling upward on end of seat to release hook and loop fastener.

2. Pull outward on hook and loop fastener strips to free them from underside of seat.

3. Fold hook and loop fastener closure strap over seat post extension to secure seat upholstery to the frame.

4. Repeat process for opposite side.

5. Return chair to upright position and push down on seat to secure it to the frame.

## Lifetime Limited Warranty

Your Drive Medical branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

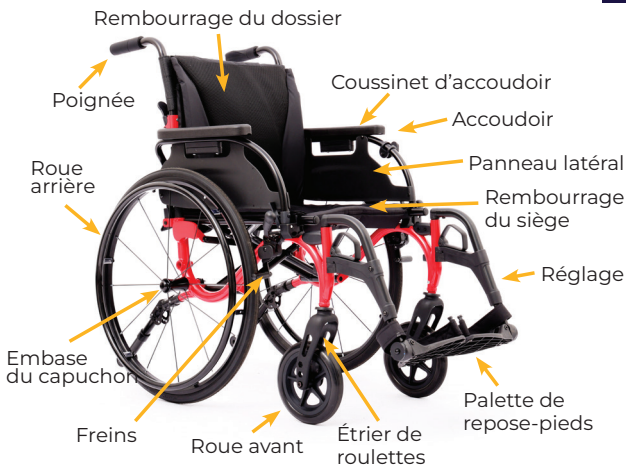
If you have a question about your Drive Medical device or this warranty, please contact an authorized Drive Medical dealer.

 **WARNING:** This product can expose you to chemicals, including di(2-ethylhexyl)phthalate (DEHP), which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).

© 2026 Medical Depot, Inc. All rights reserved.  
Drive Medical is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

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© 2025 Medical Depot, Inc. Todos los derechos reservados.