

ENDEAVOUR AXIS KIT INSTRUCTIONS


1

In order to install the swivel wheels, first you must remove the standard front wheel by unfolding the quick release lever and sliding the wheel out of the fork tips



2

Next loosen the black knob on the top of the Axis swivel wheel adaptor and unfold the hinged lid

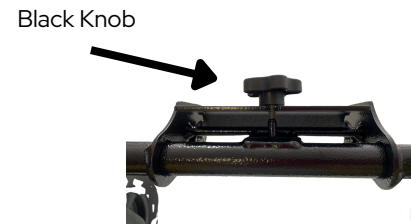

3

Now slide both forks into the Axis Swivel Wheel Adapter until the Fork Tips slide over the round cross bar



4

Finally fold the hinged flap closed and turn the black knob until snug. Give the Axis Swivel Wheel kit a tug and make sure it is on tightly.



WARNING!!!

**PLEASE ENSURE THE AXIS KIT IS PROPERLY
INSTALLED BEFORE EACH USE**