

Perry® Latex Surgical Gloves



McKesson perry latex surgical gloves provide top-notch protection against chemotherapy drugs, meeting ASTM D6978-05 standards. SureFit™ Technology prevents roll down for a secure fit during procedures. Powder-free design reduces post-op complications and sensitivity, while continuous polymer coating ensures easy donning, even with damp hands. The smooth, anti-slip finish enhances grip and precision. Sterilized with gamma radiation, these gloves uphold high hygiene and safety standards. Perfect for surgical environments with chemotherapy concerns, they are available in various sizes for an ideal fit.

Donnibility with Dermashield

Features a continuous polymer coating for excellent dry- and damp-hand donnability, allowing for easy and efficient glove changes

Powder-Free

Eliminates powder-related post-operative complications and sensitivity, offering a safer option for both medical professionals and patients

Sterilization


Sterilized using gamma radiation

Smooth Finish

Anti-slip, smooth finish designed to help enhance grip and precision during surgical procedures

SureFit™ Technology

Engineered to enhance roll down protection, ensuring a secure fit during critical procedures

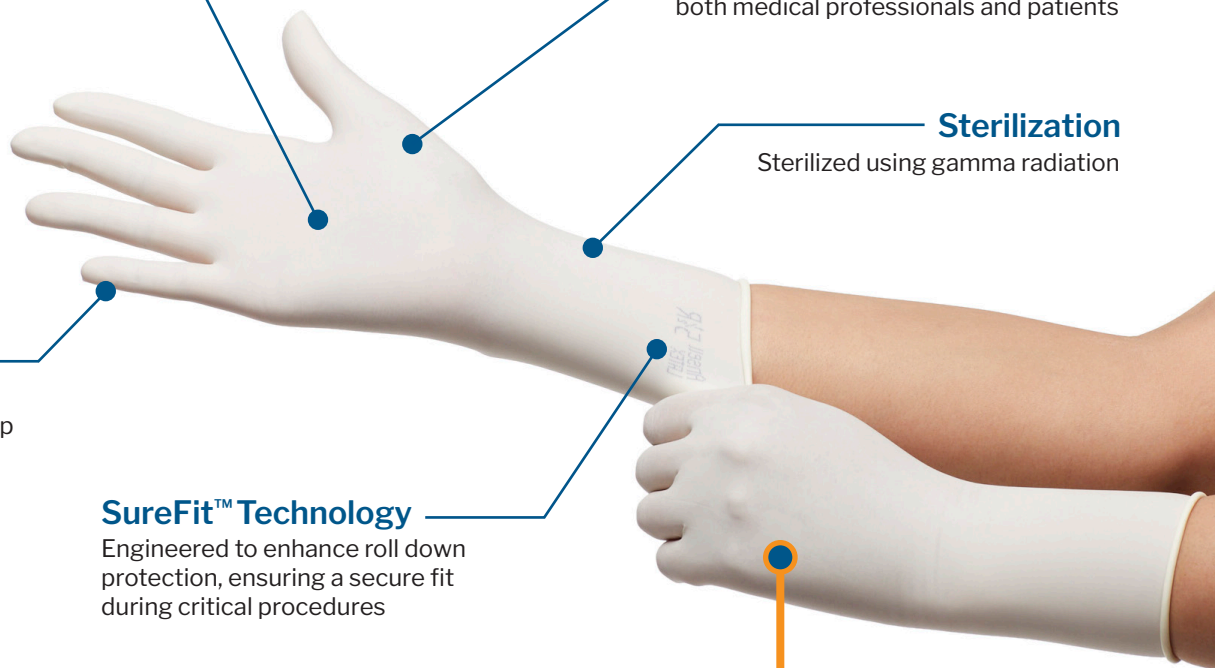


Chemo Rated

Designed to protect against chemotherapy drugs and meet ASTM D6978-05 safety standards



CHEMOTHERAPY DRUG PERMEATION RESISTANCE TESTED PER ASTM D6978-05	
Chemotherapy Drugs	McKesson-Perry Latex Surgical Breakthrough Times (BTT)
Carmustine 3.3mg/ml	DO NOT USE
Cyclophosphamide (Cytoxan) 20.0mg/ml	BBT > 240 minutes
Doxorubicin Hydrochloride 2.0mg/ml	BBT > 240 minutes
Etoposide 20.0mg/ml	BBT > 240 minutes
5-Fluorouracil, 50.0mg/ml (50,000 ppm)	BBT > 240 minutes
Paclitaxel (Taxol) 6.0mg/ml	BBT > 240 minutes
Thiotepa 10.0mg/ml	DO NOT USE
Vincristine Sulfate 1mg/ml	BBT > 240 minutes
Methotrexate 25.0mg/ml	BBT > 240 minutes
Tested for use with Chemotherapy Drugs using ASTM D6978-05. Gloves used for protection against chemotherapy drug exposure must be selected specifically for the type of drug being used.	
*Not recommended for use with Carmustine and Thiotepa. Meet or Exceeds ASTM D6978-05	

Description	E1 #	MFR #	Size	QTY
 McKesson Perry® Latex Surgical Gloves	1044704	20-1060N	6	1 pr/pch, 50 pch/bx, 4 bx/cs
	1044705	20-1065N	6.5	1 pr/pch, 50 pch/bx, 4 bx/cs
	1044706	20-1070N	7	1 pr/pch, 50 pch/bx, 4 bx/cs
	1044707	20-1075N	7.5	1 pr/pch, 50 pch/bx, 4 bx/cs
	1044708	20-1080N	8	1 pr/pch, 50 pch/bx, 4 bx/cs
	1044709	20-1085N	8.5	1 pr/pch, 50 pch/bx, 4 bx/cs
	1044710	20-1090N	9	1 pr/pch, 50 pch/bx, 4 bx/cs

Satisfaction Guaranteed

For complete details, please visit mms.mckesson.com/mckesson-brands.

Never Compromise on Care™

With McKesson Brands, you'll never have to compromise. We deliver quality products at competitive prices, backed by our 100% satisfaction guarantee. We have decades of experience in the market, and we offer you support and work to help ensure you have the products you need, when you need them—making it even easier for you to provide the best care for your patients.

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers and have not been independently verified by McKesson Medical-Surgical or its affiliates ("McKesson"). McKesson is not responsible for errors or omissions in the product information.

The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.

Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.