11080 • Pap Perfect® Plastic Spatula

Instruction for Use (English)

DEVICE DESCRIPTION

The Pap Perfect® Plastic Spatula gently rubs the mucosa, ensuring a rich sample from ectocervix. The polypropylene plastic is inert and will release collected cells easily onto the slide for Pap Smear or in the vial for Liquid Cytology. The Pap Perfect Plastic Spatula is a **Single use, Non-sterile** device. Discard after use.

CAUTION

- U.S. Federal law restricts this device to sale by or on the order of a physician.
- For use by trained medical staff only.
- For single use only. Do not reuse, reprocess, or re-sterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or re-sterilization may create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

PLEASE NOTE: If a serious incident is suspected from using the Pap Perfect Plastic Spatula, report the details of the incident to CooperSurgical via phone number +1 203-601-5200 Ext 3100 or by email at ProductSurveillance@coopersurgical.com and to the local Health Authority in your country. A serious incident may have caused or contributed to a death, a delay in a procedure which resulted in death or serious injury, or a malfunction that could have caused an adverse event.

INDICATIONS FOR USE

The Pap Perfect Plastic Spatula is used to collect cell samples from the ectocervix for Pap smear or Liquid Cytology. (The endocervical sample is usually collected with a Cytobrush® Plus.)

CONTRAINDICATIONS

Do not use on patients suspected of infection, who demonstrate profuse bleeding or are suspected of carrying sexual transmitted diseases.

INTENDED USERS

The intended users of the Pap Perfect Plastic Spatula are healthcare professionals and other trained medical personnel in a professional environment such as a hospital, clinic, or doctor's office.

PATIENT POPULATION

The device can be used by any patient women between the ages of 21 years old up to 65 years.

SAMPLING INSTRUCTIONS

1. Gently fixate the spatula tip in the center of the portio vaginalis.
2. Rotate the spatula slowly one (1) full turn.
3. Withdraw the Pap Perfect Plastic Spatula.

PREPARATION OF SAMPLE FOR PAP SMEAR CYTOLOGY

1. Smear the spatula evenly over a clean glass slide leaving room for the endocervical sample.
2. Fix immediately with cytology spray fixative or 95% ethanol.

PREPARATION OF LIQUID CYTOLOGY

Consult your local laboratory for detailed instructions.

EXPLANATION OF SYMBOLS

Reorder Number

Batch Code

Do not re-use

Do not use if package is damaged

Non-sterile

Consult instructions for use

Date of manufacture

Manufacturer

Authorized Representative in the European Community

Not made with natural rubber latex

Product conforms to the Medical Device Directive 93/42/EEC

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© 2021 CooperSurgical, Inc.
CooperSurgical Distribution B.V.