


REDI-CUT® DISPOSABLE SAFETY SCALPEL

Safety Sheath Activation Instructions for Use

RETRACT SAFETY SHEATH FOR USE


STEP 1:

From the safe position, place thumb on sheath button. Press down and pull back to retract sheath until it **clicks**.

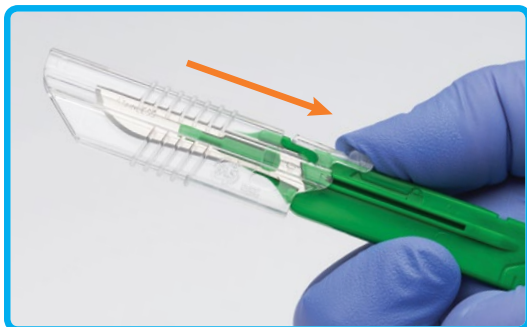


STEP 2:

Once sheath is fully retracted, scalpel is ready for use.

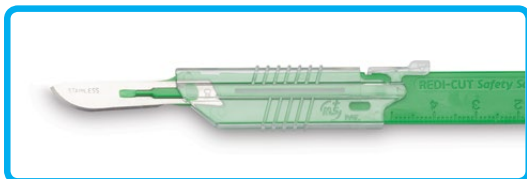

RETURN TO SAFE POSITION


Place thumb on sheath button, push down and forward until it **clicks**. The scalpel is now in the **SAFE** position.

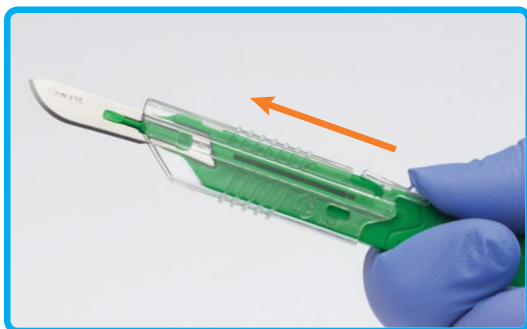

LOCK FOR DISPOSAL

ONE-HAND ACTIVATION:

From the **SAFE** position, place thumb on sheath button and push forward with force until it **clicks**. **FINAL LOCK** is activated.

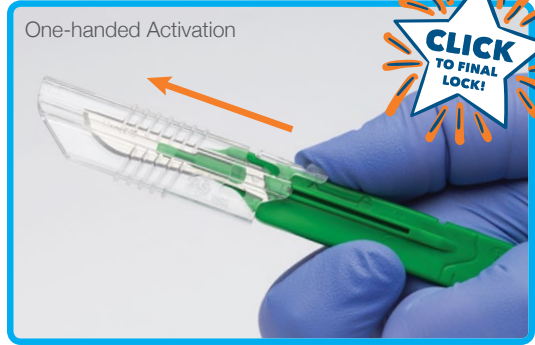

TWO-HANDED ACTIVATION:

From the **SAFE** position, place thumb of one hand on sheath button, and place thumb and index finger of second hand on the bottom ridges of the sheath. Push sheath button forward with force while assisting with your second hand until it **clicks**. **FINAL LOCK** is activated.



Arrows line up to indicate **FINAL LOCK**. Scalpel can now be disposed of responsibly.

