

ITEM #S: KN2100-BK (Black)
KN2100-OR (Orange)

KNEE WALKER W/ SHOCK ABSORBERS & TOTE BAG

WEIGHT CAPACITY 300 LBS



PARTS DIAGRAM



1. Handles
2. Brake with Parking Locker
3. Steering Column
4. Carry Bag
5. Triangle Knob
6. PU Pad
7. Knob + Tenon Bolt
8. Front Wheel
9. Back Wheel
10. Clip

PARTS LIST



2 x Clips



2 x Tube Plugs



2 x Knob & Tension Bolts



4 x Triangle Knobs



2 x Five Star Knobs

INSTALL BACK WHEELS

Step 1. Install the back wheels

Take out the main frame (Figure 1). Align hole and lock the back wheel sets with clips (Figure 2 & 3). The distance between main frame and back wheels, there are 4 levels could be adjusted by clips. Ensure it's securely installed.



Figure 1



Figure 2


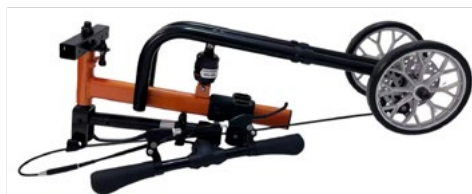


Figure 3

INSTALL FRONT WHEELS

Step 2. Install the front wheels

Insert the tube plug into the square tube of the main frame on the left and right.

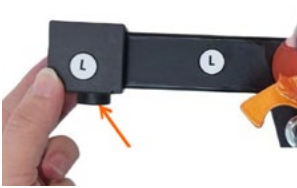

Turn the front wheel tube downward, with the push button facing the mainframe.



Insert into square tube of main frame which will make the push button pop out.



Insert triangle knob into the front hole.



INSTALLING STEERING COLUMN

Step 3. Install the steering column



Push the steering tiller upward until it's positioned straight vertically.



Under the knob



Lock the five star knob. Ensure the knob is tightly secured in place.



The five star knob is on the right side when you use the knee walker.

INSTALL HANDLE BARS

Step 4. Install the handle bars



Insert the handlebars into the steering column.



Align holes and insert triangle knob through the steering column.

INSTALL KNEE PAD

Step 5: Install the knee pad

- Insert the knee pad to desired height (Figure A).
- Insert tension bolt and tight it with knob securely.(Figure B)
- Move knee pad forward or backward, adjust it to your desired distance then tighten with triangle knob. (Figure C)



A



B



C

INSTALL BAG

Step 6: Install the bag

Hang the bag from the steel plate (Figure A &B), and then attach the Velcro on the bottom to make it firm enough to keep it secure when using the knee walker. Figure C



CHECK BRAKES

Step 7: Check the brake

Squeeze the brake lever and push down to engage brake lock.
(Figure A and B) Squeeze the brake lever again to release the brake lock.

*If your brakes become loose or need to be adjusted, please contact your provider for service.



A



B

ADJUSTING HEIGHT OF HANDLE BARS

Remove five star knob.

Extend handlebars up to desired height, align holes in tiller (Figure A).

Insert star screw through the steering tiller.

Tighten knob securely. Handlebars should be set at wrist level to keep your back straight (Figure B).



A



B

STORAGE

Remove the five star knob under the bag and the tiller to fold into compact unit for easy storage or transportation (Figure A).



A

USING YOUR KNEE WALKER

1. Place your hands on the handlebars
2. Place the injured leg comfortably resting on the center of the knee pad.
3. Release the brake to move yourself forward or backward slowly with small steps with your other leg, keeping the knee of your injured foot centered on the knee pad.
4. The knee walker can make turns by simply moving the handlebars with a minimum amount of effort. Turns should only be performed while at a slow rate of movement. To turn left, rotate handlebars to the right. To turn right, rotate handlebars to the right.
5. With practice, you should be able to move forward without deviating to the opposite side.

CLEANING/MAINTENANCE

- Clean with damp cloth and non-abrasive cleanser.
- Clean wheels with warm water and a mild cleanser Dry with a clean rag.
- Ensure that all attached hardware and seat lever are secure at all times.
- Inspect the Knee Walker periodically to ensure the brakes, seat lever, steering, and wheels are functioning properly and that all parts are secure.
- Replace any broken, damaged or worn items immediately.
- If handgrip is loose, DO NOT USE. Replace it or contact us immediately.
- Ensure that the brakes are working correctly. Over time the brake cable may stretch with use, they can be adjusted by yourself or a qualified dealer. Failure to do this may result in injury to the user.

LIMITED LIFETIME WARRANTY

Limited Lifetime Warranty against manufacturing defects. This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications.

Normal wear and tear on non-durable components, such as rubber accessories and casters are not covered under this warranty. These items are subject to normal wear and need periodic replacement.

For Warranty Service, it is recommended but not required, that the product be returned to the Rhythm Healthcare, LLC dealer through whom it was originally purchased. However, all Rhythm Healthcare, LLC dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare, LLC, prior authorization will be required. In the event of a defect covered by this warranty, Rhythm Healthcare, LLC will determine if the product will be repaired or replaced.



W: www.rhythmhc.com
E: contactus@rhythmhc.com

V3: 8/16/24