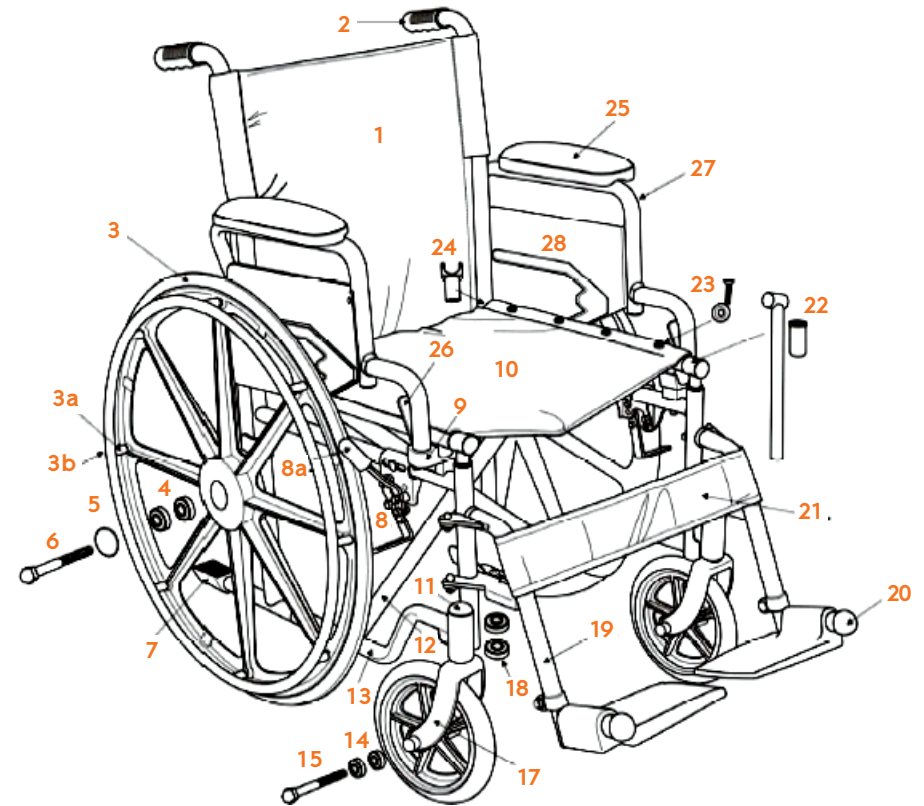


- | | |
|-------------------------------------|---|
| 1. Back Upholstery (16", 18", 20") | 14. Caster Bearings |
| 2. Handgrip - TRHG | 15. Front Axle Bolt |
| 3. Rear Wheel | 16. Front Caster |
| 3a. Handrim Screw & Spacers | 17. Fork |
| 3b. Handrim | 18. Fork Stem Bearings |
| 4. Rear Bearings | 19. Footrest Set |
| 5. Hub Cap | 20. Front Bumper |
| 6. Rear Axle Bolt | 21. Calf Strap |
| 7. Tipping Sleeve | 22. Slide Tube & Bushing |
| 8. Brake Assembly | 23. Seat Upholstery Bolts |
| 8a. Brake Tip | 24. Seat Rail Guide |
| 9. Seat/Arm Saddle | 25. Arm Pad |
| 10. Seat Upholstery (16", 18", 20") | 26. Arm Release Lever |
| 11. Dust Cap | 27. Desk Arm Left
Desk Arm Right |
| 12. Cross Frame | 28. Left Side Guard
Right Side Guard |
| 13. Side Frame | |


DESCRIPTION
DIAGRAM #
RHYTHM ITEM #

DESCRIPTION	DIAGRAM #	RHYTHM ITEM #
Seat and Back Upholstery Set	1,10	(16") P170-1-004-16 (18") P170-1-004-18 (20") P170-1-004-20
Rear Wheel Kit	3, 3a, 3b, 4	P170-1-006-1
Brake Assembly	8, 8a	(Left) PWCBRK4018-L (Right) PWCBRK4018-R
Front Caster	14, 16	P170-1-005
Front Fork	17	PFKK1
Footrest Set	19,20	PSFR
Rear Seat Rail Guide	24	PSG-R
Arm Pad	25	PWCAP-D